



RODIN[®]
3D Resin

A Pac-Dent Brand

SOFT RELINE KIT

Superior Bonding with Universal Applications



Soft Reline is the first universal reline material designed for both traditional acrylic and 3D-printed denture bases.

Soft Reline Kit

BENEFITS & FEATURES



Exceptional Adhesion and Tear Strength:

Provides unmatched adhesion and tear strength for 3D denture base polymers and traditional PMMA materials to ensure a secure and rip free bond



Superior Performance:

Advanced formulation exceeds the capabilities of traditional soft reline materials and is designed for enhanced durability and comfort, ensuring a long-lasting and comfortable fit.



Ease of Workflow:

Streamlines the denture reline process, making it efficient and to get patients in and out, minimizing chair time.



Esthetic and Patient Comfort:

Designed to blend seamlessly with the denture base, providing a natural look and feel. The result is a functional, comfortable, and visually appealing soft reline that patients can wear with confidence.



APPLICATIONS



3D Printed Dentures



PMMA Dentures



Acrylic Dentures

	Rodin Soft Reline	Acrylic Soft Reline
Configuration	Automix closed-cartridge delivery; consistent, clean, minimal waste	Powder & liquid mix; variable ratios, messy, higher waste
Digital Integration	Optimized for 3D-printed denture bases; seamless digital workflow	Not designed for digital workflows
Longevity	Lasts up to 1 year or more	3-6 months
Durability	Exceptional tear strength, long-term resilience	Prone to cracking and becoming brittle
Comfort	Maintains elasticity and softness over time	Hardens and loses comfort quickly
Odor/Taste	Neutral, no offensive odor or taste	Often associated with unpleasant odor/taste over time



Soft Reline Kit

WORKFLOW




1. Roughen Intaglio Surface

The intaglio surface and borders of both dentures were aggressively roughened and beveled. The intaglio—including flange edges—was polished with a brown wheel to create an ideal bonding



2. Clean & Apply Primer

Each denture was cleaned thoroughly and dried. VPS Primer was applied evenly to all intaglio surfaces, allowing proper drying time per IFU guidelines.

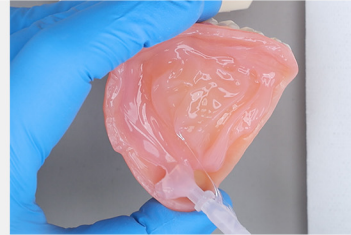

4. Soft Reline Application

Soft Reline was dispensed evenly using the automix cartridge. The dentures were seated intraorally for functional adaptation and allowed to set for 5 minutes.



5. Trimming & Final Fit

After removal, excess material was trimmed with a scalpel. The dentures were allowed to fully set extraorally before final delivery and patient evaluation.



1

Soft Reline VPS Reline Cartridge, 50 mL

Silicone-based reline material delivers excellent bond strength and tear resistance for 3D-printed, PMMA milled, and traditional acrylic dentures.

2

Soft Reline VPS Primer, 10 mL bottle

A bonding primer for VPS-based reline materials that enhances adhesion and durability between Soft Reline and denture bases.

3

Yellow 1:1 Wide Mouth Mixing Tips Refill, 6 pk

Wide mouth mixing tips designed for efficient and consistent dispensing of 1:1 ratio materials.

4

Polishing Wheel & Cone Finishing Stone

Gently smooths reline material without abrasion, delivering consistent, comfortable results.

5

Applicator Swabs Refill, 6 pk

Used to apply primer for improved adhesion between VPS-based reline material and denture bases.

Soft Reline Kit



Dr. Russell Schafer, DDS

I've been using Soft Reline since it was first released and have been very pleased with the results. It is easy to apply and has been holding up well in my patients' mouths. The neutral taste means that patients don't complain during the procedure and are happy with the end result as well. I also like the fact that it bonds to a wide variety of materials meaning that I can keep a more limited inventory and don't need to worry about waste.



Item#

Product Description

24006

Soft Reline Kit

1x Reline Silicone Cartridge, 50mL
 1x VPS Adhesive Primer, 10mL
 1x Brown Polishing Wheel
 1x Tapered Cone Finishing Stone

6x Yellow 1:1 Wide Mouth Mixing Tips
 6x Applicator Swabs



24007

50ml Reline Silicone Cartridge



24008

VPS Adhesive Primer - 10ml



24009

VPS Remover- 10ml



240010

Tapered Cone Finishing Stone



240011

Brown Polishing Wheel



240012

VPS Adhesive Primer - 10ml



240013

Yellow 1:1 Wide Mouth Mixing Tips Refill, 48 pk

BUY 2 Soft Reline Kits, GET 1 FREE!

To qualify and receive your promotional goods please submit a valid proof of purchase to freegoods@pac-dent.com. Invoices must be submitted via email to qualify. Cannot Be Combined With Any Other Offer. No Cash Value. Same or Lesser Item(s) Applied. Offers are valid only at participating dealers in the continental US, excluding AK and HI. Promotions are subject to change or cancellation without notice. Please allow 6-8 weeks for delivery. Ends 12/31/2025



Streamline Your Efficiency

Grab a Free Sample



Pac-Dent, Inc.
 670 Endeavor Circle Brea, CA 92821
www.pac-dent.com

(909) 839-0888 | rodin@pac-dent.com | www.rodin-3d.com